

| [a]: Cleaners | [b]: Wipes | [c]: Sprays | [d]: Solutions | [e]: Devices | [f]: Powder | | |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------|
| a1: MetriZyme a2: Tristel Pre-Clean Wipes a3: Liquinox a4: Revital-OX Enzymatic Detergent a5: MetriSponge a6: Prolystica 2X Concentrate Enzymatic Cleaner a7: Endozime and Endozime Sponge a8: klenzyme | b1: CLEANISEPT® WIPES b2: mikrozyd® AF Wipes Jumbo b3: PROTEX™ DISINFECTANT Wipes b4: Sani-Cloth® Plus b5: SONO™ ULTRASOUND WIPES b6: Tristel Surface Wipes b7: Tristel Sporidical Wipes b8: Tristel Rinse Wipes b9: Clineil Universal Wipes b10: mikrozyd® Sensitive Wipes b11: Wip'Anios premium b12: ultrasound probe cleaning wipes | b13: Sani-Cloth AF3 (gray) b14: Protex Ultra Wipes b15: Sani-Cloth HB b16: CaviWipes b17: Dispatch Towels b18: Accel TB Wipes b19: CaviWipes 1 b20: Tuffle 5 b21: Sani-Cloth Active b22: Septiwipes b23: Mikorbac Tissues b24: Sani-Cloth Germicidal Wipes b25: WIP'ANIOS CLEAN'UP b26: CaviWipes XL b27: OXIVIR™ WIPES b28: Oxivir 1 Wipes | c1: Oxivir™ TM Tb c2: PI-SPRAY II c3: Surfa'safe c4: TRANSEPTIC c5: PROTEX™ DISINFECTANT SPRAY c6: Tristel Solo c7: Tristel Duo c8: IODOCLEAN c9: Protex Spray c10: CaviCide c11: T-Spray c12: Indican Form c13: Transeptic Spray c14: CAVICIDE 1 c15: OXIVIR 1 c16: WIP'ANIOS SPOR'ACTIV c17: Accel TB Liquid | d1: CIDEX OPA d2: Cidex Activated Dialdehyde Solution d3: Minncare® Cold Sterilant d4: Ster-Bac d5: Triacid-N d6: Revital-Ox™ Resert High Level Disinfectant d7: ggigasept® PAA concentrate d8: DESCOTON extra d9: gigasept® FF(neu) d10: ANIOXYDE 1000 d11: SALVANIOS pH10 d12: Cavicide Liquid d13: Metricide | d14: Metricide 28 d15: Metricide OPA Plus d16: Cidex Plus d17: Gigasept AF d18: Osvan d19: Neojodin d20: Milton d21: hibitane d22: Sterihyde d23: Metricide 14 d24: Sekusept plus d25: Wavicide-01 d26: SALVANIOS pH7 d27: Minncare liquid disinfectant | e1: TrophonSonex-HL (Used with Trophon/Trophon2) e2: Germitec Antigermix UV-C e3: Vaporized Hydrogen Peroxide (Used with V-PRO Low Temperature Sterilization System) | f1: Rely+On PeraSafe |

| Probe | [a]: Cleaners | [b]: Wipes | [c]: Sprays | [d]: Solutions | [e]: Devices | [f]: Powders |
|-------------------------------------------------------------------------------------------------------------------------------|----------------------------|-------------------------------------------------------------------------------------------|----------------------------------------------------|----------------------------------------------------------------------------|--------------|--------------|
| C5-1s/C5-1E/C5-1U/C5-1/C5-2/C5-2s/C5-2E/C6-2E/C6-2/C6-2s/C6-2P/C11-3E/C11-3s/C11-3U/C11-3 | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c16, c17 | d1, d4, d5, d6, d7, d8, d9, d10, d11, d12, d26 | e1, e2 | |
| C6-2Gs/C6-2GE/C6-2GU | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c16, c17 | d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27 | e1, e2 | |
| C7-3E/3C1/3C1s/3C1P/6C2/6C2P/6C2s/35C20EA/35C50EA/35C50EB/35C50P/65C15EAV/65C15EA | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27 | e1, e2 | |
| 3C5/3C5s/3C5A/3C5P | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27 | e1, e2 | |
| SC5-1U/SC5-1E | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c14, c15, c16, c17 | d1, d4, d6, d12, d15 | e1, e2 | |
| SC6-1U/SC6-1E/SC6-1s/SC5-1NE/SC5-1Ns/SC5-1N | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c16, c17 | d1, d4, d5, d6, d7, d10, d11, d12, d15, d26 | e1, e2 | |
| SC8-2U/SC8-2E/SC8-2s | a1 | b25 | c4, c16 | d1 | e1 | |
| 65EC10EA/65EC10EB/65EC10EC | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d12, d13, d14, d15, d27 | e1, e2, e3 | |
| 65EC10ED | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d2, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26 | e1, e2, e3 | |
| C4-1/C4-1U/C4-1s | a3, a4, a5, a6, a7 | b4, b5, b13, b14, b15, b16, b17, b18 | c2, c9 | d1, d2, d6, d12, d13, d14, d15 | | |
| C9-3Ts | | b16 | | d1, d6, d12, d13, d14, d15 | e1 | |
| LAP13-4Cs | | | | d2 | e3 | |
| L7-3/L7-3s/L7-3E/7L4/7L4s/7L4A/7L4P/7L4B/7L4Bs/7L4BP/7L5/7L5P/7L5s/7L6/7L6s/75L38EA/75L38EB/75L53EA/75L60EA/75L38P/10L4/10L4s | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27 | e1, e2 | |

P/N: 046-008412-00 (26.0)



| [a]: Cleaners | [b]: Wipes | [c]: Sprays | [d]: Solutions | [e]: Devices | [f]: Powder | | |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------|
| a1: MetriZyme a2: Tristel Pre-Clean Wipes a3: Liquinox a4: Revital-OX Enzymatic Detergent a5: MetriSponge a6: Polystica 2X Concentrate Enzymatic Cleaner a7: Endozime and Endozime Sponge a8: klenzyme | b1: CLEANISEPT® WIPES b2: mikroziid® AF Wipes Jumbo b3: PROTEX™ DISINFECTANT Wipes b4: Sani-Cloth® Plus b5: SONO™ ULTRASOUND WIPES b6: Tristel Surface Wipes b7: Tristel Sporicidal Wipes b8: Tristel Rinse Wipes b9: Clinell Universal Wipes b10: mikroziid® Sensitive Wipes b11: Wip'Anios premium b12: ultrasound probe cleaning wipes | b13: Sani-Cloth AF3 (gray) b14: Protex Ultra Wipes b15: Sani-Cloth HB b16: CaviWipes b17: Dispatch Towels b18: Accel TB Wipes b19: CaviWipes 1 b20: Tuffle 5 b21: Sani-Cloth Active b22: Septiwipes b23: Mikorbac Tissues b24: Sani-Cloth Germicidal Wipes b25: WIP'ANIOS CLEAN'UP b26: CaviWipes XL b27: OXIVIR™ WIPES b28: Oxivir 1 Wipes | c1: Oxivir™ ^{TMMC} Tb c2: PI-SPRAY II c3: Surfa'safe c4: TRANSEPTIC c5: PROTEX™ DISINFECTANT SPRAY c6: Tristel Solo c7: Tristel Duo c8: IODOCLEAN c9: Protex Spray c10: CaviCide c11: T-Spray c12: Indican Form c13: Transeptic Spray c14: CAVICIDE 1 c15: OXIVIR 1 c16: WIP'ANIOS SPOR'ACTIV c17: Accel TB Liquid | d1: CIDEX OPA d2: Cidex Activated Dialdehyde Solution d3: Minncare® Cold Sterilant d4: Ster-Bac d5: Triacid-N d6: Revital-Ox™ Resert High Level Disinfectant d7: ggigasept® PAA concentrate d8: DESCOTON extra d9: gigasept® FF(neu) d10: ANIOXYDE 1000 d11: SALVANIOS pH10 d12: Cavicide Liquid d13: Metricide | d14: Metricide 28 d15: Metricide OPA Plus d16: Cidex Plus d17: Gigasept AF d18: Osvan d19: Neojodin d20: Milton d21: hibitane d22: Sterihyde d23: Metricide 14 d24: Sekusept plus d25: Wavicide-01 d26: SALVANIOS pH7 d27: Minncare liquid disinfectant | e1: TrophonSonex-HL (Used with Trophon/Trophon2) e2: Germitec Antigermix UV-C e3: Vaporized Hydrogen Peroxide (Used with V-PRO Low Temperature Sterilization System) | f1: Rely+On PeraSafe |

| | Probe | [a]: Cleaners | [b]: Wipes | [c]: Sprays | [d]: Solutions | [e]: Devices | [f]: Powders |
|-------------------------|-----------------------------------------------------------------------------------------------------------------|--------------------------------------|-------------------------------------------------------------------------------------------|----------------------------------------------------|----------------------------------------------------------------------------|--------------|--------------|
| Linear | 65EL60EA/6LE7/6LE7s/6LE7P/6LE5V/6LE5Vs/6LE5VP/7LT4/7LT4s/7LT4P/7LT4E/50L60EAV/65L50HAV/75L50EAV/75LT38EA | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d12, d13, d14, d15, d27 | e1, e2, e3 | |
| | L9-3U/L9-3E/L9-3s/L12-3/L12-3E/L13-3s/L13-3/L12-4/L12-4s/L14-6N/L14-6Ns/L14-6NP/L14-6NE/L14-6WE/L14-6Ws/L14-6WU | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c16, c17 | d1, d4, d5, d6, d7, d8, d9, d10, d11, d12, d26 | e1, e2 | |
| | L10-3E/L10-3s | a1 | b25 | c5, c6 | | e1 | |
| | L11-3U/L11-3VNs/L13-3Ns/L14-3WU/L14-3WE | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c16, c17 | d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27 | e1, e2 | |
| | L11-4/L11-4s/L14-6/L14-6s/L14-6P/10L24EA | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c16, c17 | d1, d4, d5, d6, d7, d10, d11, d12, d15, d26 | e1, e2 | |
| | L16-4HE/L16-4Hs/L16-4HU | | | | d1, d2 | | |
| | LM14-6E/LM14-6s/LM16-4U | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27 | e1, e2 | |
| | L20-5U/L20-5s/L20-5E | | b25 | c2 | d2 | | |
| | L12-3RCs/L12-3VNs | | b1, b4, b5, b11, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c3, c14, c15, c16, c17 | d12 | e1 | |
| | L13-3WE/L13-3WU | | b25 | c7, c16 | | | |
| L14-5sp | | b4, b5, b13, b14, b15, b16, b17, b18 | c2, c9 | d1, d2, d6, d12, d13, d14, d15 | e1 | | |
| L14-5WU/L14-5WE/L14-5Ws | | b4, b16 | c2, c11 | d1, d2 | | | |
| Phased | P4-2/P4-2s/P4-2E/P4-2NE/P4-2Ns/P10-4E/P10-4s/P10-4U/2P2/2P2s/2P2P | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27 | e1, e2 | |
| | P7-3/P7-3s/P7-3E/P7-3U/P7-3P | a1 | b3, b25 | c2, c4, c5, c6, c16 | d1 | e1 | |
| | P12-4/P12-4s | a1 | b5, b25 | c1, c2, c5, c16 | d1, d2, d3, d4, d5, d27 | e1 | |
| | SP5-1U/SP5-1s/SP5-1E/SP5-1 | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d4, d5, d6, d7, d10, d11, d12, d15, d26 | e1, e2 | |
| | SP5-1Ns/SP5-1N | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c16, c17 | d1, d4, d5, d6, d7, d8, d9, d10, d11, d12, d26 | e1, e2 | |

| [a]: Cleaners | | [b]: Wipes | | [c]: Sprays | | [d]: Solutions | | [e]: Devices | [f]: Powder |
|-------------------------------------------------------|---------------------------------------------|-----------------------------------------|---------------------------------------|-------------------------------------------------------|------------------------------------------|-----------------------------------------------------------------------------------------------|-----------------------------|--------------|-------------|
| a1: MetriZyme | b1: CLEANISEPT® WIPES | b13: Sani-Cloth AF3 (gray) | c1: Oxivir™ TM Tb | d1: CIDEX OPA | d14: Metricide 28 | e1: TrophonSonex-HL (Used with Trophon/Trophon2) | f1: Rely+On PeraSafe | | |
| a2: Tristel Pre-Clean Wipes | b2: mikrozyd® AF Wipes Jumbo | b14: Protex Ultra Wipes | c2: PI-SPRAY II | d2: Cidex Activated Dialdehyde Solution | d15: Metricide OPA Plus | | | | |
| a3: Liquinox | b3: PROTEX™ DISINFECTANT Wipes | b15: Sani-Cloth HB | c3: Surfa'safe | d3: Minncare® Cold Sterilant | d16: Cidex Plus | e2: Germitec Antigermix UV-C | | | |
| a4: Revital-OX Enzymatic Detergent | b4: Sani-Cloth® Plus | b16: CaviWipes | c4: TRANSEPTIC | d4: Ster-Bac | d17: Gigasept AF | e3: Vaporized Hydrogen Peroxide (Used with V-PRO Low Temperature Sterilization System) | | | |
| a5: MetriSponge | b5: SONO™ ULTRASOUND WIPES | b17: Dispatch Towels | c5: PROTEX™ DISINFECTANT SPRAY | d5: Triacid-N | d18: Osvan | | | | |
| a6: Polystica 2X Concentrate Enzymatic Cleaner | b6: Tristel Surface Wipes | b18: Accel TB Wipes | c6: Tristel Solo | d6: Revital-Ox™ Resert High Level Disinfectant | d19: Neojodin | | | | |
| a7: Endozime and Endozime Sponge | b7: Tristel Sporicidal Wipes | b19: CaviWipes 1 | c7: Tristel Duo | d7: ggigasept® PAA concentrate | d20: Milton | | | | |
| a8: klenzyme | b8: Tristel Rinse Wipes | b20: Tuffle 5 | c8: IODOCLEAN | d8: DESCOTON extra | d21: hibitane | | | | |
| | b9: Clinell Universal Wipes | b21: Sani-Cloth Active | c9: Protex Spray | d9: gigasept® FF(neu) | d22: Sterihyde | | | | |
| | b10: mikrozyd® Sensitive Wipes | b22: Septiwipes | c10: CaviCide | d10: ANIOXYDE 1000 | d23: Metricide 14 | | | | |
| | b11: Wip'Anios premium | b23: Sani-Cloth Germicidal Wipes | c11: T-Spray | d11: SALVANIOS pH10 | d24: Sekusept plus | | | | |
| | b12: ultrasound probe cleaning wipes | b24: WIP'ANIOS CLEAN'UP | c12: Indican Form | d12: Cavicide Liquid | d25: Wavicide-01 | | | | |
| | | b25: WIP'ANIOS CLEAN'UP | c13: Transeptic Spray | d13: Metricide | d26: SALVANIOS pH7 | | | | |
| | | b26: CaviWipes XL | c14: CAVICIDE 1 | | d27: Minncare liquid disinfectant | | | | |
| | | b27: OXIVIR™ WIPES | c15: OXIVIR 1 | | | | | | |
| | | b28: Oxivir 1 Wipes | c16: WIP'ANIOS SPOR'ACTIV | | | | | | |
| | | | c17: Accel TB Liquid | | | | | | |

| Probe | | [a]: Cleaners | [b]: Wipes | [c]: Sprays | [d]: Solutions | [e]: Devices | [f]: Powders |
|-------------|-------------------------------------------------------------------|-----------------------------------|--------------------------------------------------------------------------------------------------|-----------------------------------------------------------|--------------------------------------------------------------------------|-------------------|--------------|
| Phased | LFP5-1s/LFP5-1U | | b1, b4, b5, b11, b16, b18, b19, b24, b26, b27, b28 | c1, c3, c14, c15, c17 | d12 | | |
| | P8-2s/P8-2 | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c16, c17 | d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27 | e1, e2 | |
| Endo-cavity | V11-3/V11-3BE/V11-3B/V11-3WE/V11-3Ws/V11-3E/V11-3s | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d10, d11, d12, d13, d14, d15, d26, d27 | e1, e2, e3 | |
| | V11-3HU/V11-3HE/V11-3Hs/V11-3HV/V11-3HB/V11-3HBs | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c16, c17 | d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27 | e1, e2 | |
| | V10-4/V10-4s/V10-4B/V10-4Bs/V10-4BP/V10-4P/6CV1/6CV1s/6CV1P | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d2, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26 | e1, e2, e3 | |
| Pencil | CW2s/CW5s/CW5 | | | | d1, d2, d4, d5 | | |
| Biplane | CB10-4/CB10-4P/CB10-4A/CB10-4E/6LB7/6LB7s/6LB7P/6LB7E/65EB10EA | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d12, d13, d14, d15, d27 | e1, e2, e3 | |
| | ELC13-4U | | b1, b4, b5, b11, b16, b18, b19, b24, b26, b27, b28 | c1, c3, c7, c14, c15, c17 | d1, d12, d15 | | |
| 4D | 4CD4/4CD4s/D6-2E | | | | d1, d2, d4 | | |
| | D7-2/D7-2E/D7-2s/SD8-1E/SD8-1s/D6-2/D6-2P/D6-2EA/D6-2B | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c4, c5, c6, c7, c8, c14, c15, c16, c17 | d1, d2, d3, d4, d5, d6, d10, d11, d12, d13, d14, d15, d26, d27 | e1 | |
| | D6-2NE/DL14-3U | | | | d1, d18, d19, d20, d21, d22 | e1 | |
| | D8-2E/D8-2U | | b10, b16, b19, b20 | c2, c10 | d1, d9 | e1 | f1 |
| | D8-4U | | b4, b10, b16, b19, b20 | c2, c10, c11 | d1, d2, d16, d17 | | f1 |
| | DE10-3/DE10-3E/DE10-3U/DE10-3s | | b10, b16, b19, b20, b21 | c2, c10, c11 | d1, d2, d16, d17 | e1, e2 | f1 |
| | DE11-3U/DE11-3P/DE11-3s/DE11-3E/DE11-3/DE11-3WE/DE11-3Ws/DE11-3WU | a1, a2, a3, a4, a5, a6, a7 | b1, b2, b3, b4, b5, b6, b7, b8, b9, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28 | c1, c2, c3, c5, c7, c8, c14, c15, c17 | d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27 | e1, e3 | |
| | DE10-3WU/DE10-3WE | a8 | b1, b12, b15, b16, b20, b22, b23, b24 | c2, c11, c12, c13 | d1, d2, d15, d16, d17, d23, d24, d25 | e1, e2 | f1 |

Active Ingredients of the Cleaner

| Cleaner | Active Ingredients |
|-----------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------|
| MetriZyme/MetriSponge: | Proteinase subtilisin |
| Revital-OX Enzymatic Detergent: | Citric acid, Triethanolamine, Ethanolamine, Ethoxylated coconut oil alkyl amine, Subtilisins (proteolytic enzymes), Glycerine |
| Endozime and Endozime Sponge: | Subtilisins (proteolytic enzymes) |
| Prolystica 2X Concentrate Enzymatic Cleaner/klenzyme/ Liquinox/Tristel Pre-Clean Wipes: | enzymatic detergent |

Active Ingredients of the Disinfectant

| Disinfectant | Active Ingredients |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------|
| Tristel Surface Wipes/Tristel Trio Wipes System/Tristel Jet/Tristel Duo/Tristel Sporidical Wipes: | chlorine dioxide |
| IODOCLEAN | sodium thiosulfate and excipients |
| Ster-Bac/PI-SPRAY II/mikrozyd® Sensitive Wipes/Clinell Universal Wipes/Sani-Cloth HB/Sani-Cloth Active/T-Spray/Mikorbac Tissues/Sani-Cloth Germicidal Wipes/SALVANIOS pH7/SALVANIOS pH10/Protex Ultra Wipes/CaviWipes XL/CAVICIDE 1: | Quaternary Ammoniums |
| Tristel Rinse Wipes: | deionized water |
| Cidex OPA: | 0.55% Ortho-phthlaldehyde |
| DESCOTON extra/Wavicide-01: | glutaraldehyde |
| Oxivir™/Tb: | 0.5% hydrogen peroxide |
| Tristel Solo: | polyhexamethylbiguanide |
| Sani-Cloth® Plus: | n-Alkyl dimethyl benzyl ammonium chloride n-Alkyl ethylbenzyl ammonium chloride |
| CaviWipes: | isopropanol, ethylene glycol monobutyl ether (2-butoxyethanol), diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride, water |
| CaviWipes 1: | isopropanol, ethanol, ethylene glycol monobutyl ether (2-butoxyethanol), didecyl dimethyl ammonium chloride, water |
| Dispatch Towels: | Sodium hydroxide, Sodium metasilicate, Sodium hypochlorite |
| SONO™ ULTRASOUND WIPES: | octyl decyl dimethyl ammonium chloride; dioctyl dimethyl ammonium chloride; didecyl dimethyl ammonium chloride; dimethyl benzyl ammonium chloride |
| Minnicare® Cold Sterilant/ Minnicare liquid disinfectant: | 22% Hydrogen Peroxide, 4.5% Peroxyacetic Acid |
| mikrozyd® AF Wipes Jumbo: | 25% ethanol; 35% propan-1-ol |
| CLEANISEPT® WIPES: | 0.25g didecyl dimethyl ammonium chloride 0.5g quaternary ammonium compounds, benzyl-C12-16-alkyldimethyl-, chlorides |
| Wip'Anios premium: | didecyl dimethyl ammonium chloride 1.4mg/g, polyhexamethylene biguanide hydrochloride 0.96mg/g |
| Cidex Activated Dialdehyde Solution: | glutaraldehyde |

| Disinfectant | Active Ingredients |
|-----------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------|
| TRANSEPTIC: | isopropyl alcohol, chlorhexidine gluconate |
| Protex Spray: | Quaternary ammonium compounds di-C8-10-alkyldimethyl, chlorides Water |
| gigasept® FF(neu): | 0.11g succindialdehyde; 0.3g dimethoxytetrahydrofuran; < 5% anionic surfactant; non-ionic surfactants; anti-corrosion compounds; fragrance |
| gigasept® PAA concentrate: | peracetic acid (5%); hydrogen peroxide: acetic acid; potassium hydroxide: corrosion inhibitor |
| Protex™ Disinfectant Spray/ Protex™ Disinfectant Wipes: | octyl decyl dimethyl ammonium chloride; dioctyl dimethyl ammonium chloride; didecyl dimethyl ammonium chloride; dimethyl benzyl ammonium chloride |
| Triacid-N: | N-Dodecylpropan-1,3-diamin; propan-2-ol; isotridecanol, ethoxylated; non-ionic detergent |
| Surfa'safe: | didecyl dimethyl ammonium chloride 1.4mg/g, polyhexamethylene biguanide hydrochloride 0.96mg/g |
| Revital-Ox™ Resert High Level Disinfectant/TrophonSonex-HL: | hydrogen peroxide |
| ANIOXYDE 1000: | 3% Hydrogen Peroxide |
| Sani-Cloth AF3: | Quaternary ammonium chlorides. |
| Metricide OPA Plus: | Ortho-phthlaldehyde |
| Metricide/Metricide 14/Metricide 28: | Glutaraldehyde |
| CaviCide: | isopropanol, ethanol, ethylene glycol monobutyl ether (2-butoxyethanol), diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride, water |
| Gigasept AF: | didecyl dimethyl ammonium chloride, glycine, aminoalkyl derivs tridecylpolyethylenglycoether. |
| Rely+On PeraSafe: | disodium carbonate, citric acid |
| Cidex Plus: | glutaraldehyde |
| Sonogel: | carbomer, polyacrylat |
| Sterihyde: | Glutaraldehyde |
| Osvan: | Ammonia benzalkonium |
| Neojodin: | povidone-iodine |
| Milton: | sodium hypochlorite |
| Hibitane: | Hlorhexidine |
| WIP'ANIOS CLEAN'UP: | Association of surfactants, excipients |
| OXIVIR 1/OXIVIR™ WIPES/Oxivir 1 Wipes/Accel TB Liquid/Accel TB Wipes: | Hydrogen peroxide |
| WIP'ANIOS SPOR'ACTIV: | Hydrogen peroxide and peroxyacetic acid mixture stabilized |

Registered disinfectants in Canada

| Drug Identification Number/Licence Number | Name |
|-------------------------------------------|--------------------------------------------|
| 74736 | V-PRO Low Temperature Sterilization System |
| 02286467 | Revital-Ox™ Resert High Level Disinfectant |
| 100935 | Trophon/Trophon2 |
| 01963996 | Metricide |

| Drug Identification Number/Licence Number | Name |
|-------------------------------------------|-------------------------------|
| 01964461 | Metricide 28 |
| 02197170 | Wavicide-01 |
| 02277484 | Minnicare liquid disinfectant |
| 02161656 | Cavicide Liquid |

Registered disinfectants in FDA region

| Registration Number | Name | Microbicidal Method | |
|---------------------|-------------|--------------------------------------------|-------------------------------------------|
| FDA 510(k) | K190103 | V-PRO Low Temperature Sterilization System | Sterilization |
| | K924434 | Cidex Activated Dialdehyde Solution | High-level disinfectant and Sterilization |
| | K030004 | CIDEX OPA | High-level disinfectant |
| | K923744 | Cidex Plus | High-level disinfectant and Sterilization |
| | K080420 | Revital-Ox™ Resert High Level Disinfectant | High-level disinfectant |
| | K103059 | Trophon | High-level disinfectant |
| | K173865 | Trophon2 | High-level disinfectant |
| | K930284 | Metricide | High-level disinfectant and Sterilization |
| | K931052 | Metricide 28 | High-level disinfectant and Sterilization |
| | K140703 | Metricide OPA Plus | High-level disinfectant |
| K914749 | Wavicide-01 | High-level disinfectant and Sterilization | |
| EPA | 9480-6 | Sani-Cloth® Plus | Low-level disinfectant |
| | 70627-56 | Oxivir™/Tb | Low-level disinfectant |
| | 9480-9 | Sani-Cloth AF3 (gray) | Low-level disinfectant |
| | 46781-8 | CAVIWIPES | Low-level disinfectant |
| | 46781-6 | CAVICIDE | Low-level disinfectant |
| | 70627-60 | OXIVIR™ WIPES | Low-level disinfectant |
| | 46781-12 | CAVICIDE 1 | Low-level disinfectant |

NOTE:

- For use of each disinfectant, please refer to the manufacturer's instructions.
- Please select the proper disinfectant for the probe of the Diagnostic Ultrasound System. The content of this document shall prevail in case of other new editions.
- Refer to local regulations for the use of each disinfectant.
- Refer to the operator's manual of the Diagnostic Ultrasound System for how to select a microbicidal method.