

TE7

Color Doppler Ultrasound System

Specification

Release V1.0



1 System Overview

1.1 Application

- Emergency
- Nerve
- Musculoskeletal
- Critical Care
- Abdomen
- Cardiology
- Obstetrics
- Gynecology
- Urology
- Smart part
- Vascular
- Others

1.2 Transducer types

- Curved array transducer
- Linear array transducer
- Phased array transducer

1.3 Imaging modes

- B-Mode
- THI and PSH™ (Phase Shift Harmonic Imaging)
- M-Mode
- Color Doppler Imaging
- Power Doppler Imaging/Directional PDI
- Pulsed Wave Doppler
- Continuous Wave Doppler
- Left Ventricular Opacification

1.4 Standard features

- B-Mode
- THI and PSH™
- M-Mode
- Color Doppler Imaging
- Power Doppler Imaging and Directional PDI
- Pulsed Wave Doppler
- iBeam™ (Spatial Compound Imaging)
- iClear™ (Speckle Suppression Imaging)
- iTouch™ (Auto Image Optimization)
- Smart Track

- Zoom/iZoom (Full Screen Zoom)
- FCI (Frequency Compound Imaging)
- B steer
- ExFOV (Extended Field of View)
- Post Processing
- Echo Boost™
- 3 or 1 active universal probe ports(optional)
- 120GB SSD
- Built-in wireless adapter
- Built-in battery
- 4 USB 3.0 ports
- Touch Gestures
- iStorage
- MedSight

1.5 Optional features

- Continuous Wave Doppler
- DICOM
- Shared Service Package
- Left Ventricular Opacification (LVO)
- iNeedle™ (Needle Visualization Enhancement)
- Cart
- Table stand/ Wall mount

1.6 Language support

- Software: English, Chinese, German, Spanish, French, Italian, Portuguese, Russian, Czech, Polish, Turkish, Norwegian, Serbian
- User manual: English, Chinese

2 Physical Specification

2.1 Dimension and weight

- Depth: 97mm
- Width: 295mm
- Height: 380mm
- Weight: Approx. 6.1Kg (three probe socket configuration, with probe holders, no peripherals or batteries) (battery weight: 0.9Kg)

2.2 Monitor

- 15-inch high resolution color LED monitor
- Resolution: 768*1024

- Viewing angle: 85° left/right; 85° up/down
 - Digital on-screen display of brightness and contrast controls
 - Frame rate (Hz): 60Hz
- 2.3 Cart (Option)**
- DCU independent tilt of 50 degrees up, 5 degrees down.
 - Dimension and Weight(with DCU)
 - Height: 1266-1556mm
 - Width: 535mm
 - Depth: 620mm
 - Weight: approx. 50Kg
 - Wheels
 - Diameter: 125mm
 - Castors (4ea): total lock and break
 - Towelette holster
 - Gel holder
 - Printer holder
 - Storage bin
- 2.4 Table stand (Option)**
- DCU independent tilt of 50 degrees up, 5 degrees down.
 - Dimension and Weight(with DCU)
 - Height: 248mm
 - Width: 196mm
 - Depth: 239mm
 - Weight: approx. 2.5 Kg
- 2.5 Built-in Wireless adapter**
- Encryption: WEP, WPA-PSK, WPA2-PSK
 - Max transfer speed: 300Mbps
 - Protocols: 802.11b: 11,5.5,2,1 Mbps; 802.11g: 54,48,36,24,18,12,9,6 Mbps; 802.11n: up to 300Mbps
- 2.6 Built-in Battery**
- Replaceable and rechargeable lithium battery
 - Light indicator
 - Full battery lasts more than 22h in standby mode
 - Empty battery recharged to full in less than 4h
 - Continuous work time: more than 2

- hours
- Lithium-Ion Battery Pack 14.8V, 5800mAh (single battery)
- 2.7 Probe port and holder**
- Probe ports: max. 3 active ports
 - Detachable probe holder: 4
- 2.8 Electrical power**
- Voltage: 100-240V~
 - Frequency: 50/60 Hz
 - Input current: 2.0A (115VAC)
- 2.9 Operating Environment**
- Ambient temperature: 0-40 °C
 - Relative humidity: 30%-85% (no condensation)
 - Atmospheric pressure: 700hPa-1060hPa
- 2.10 Storage & Transportation Environment**
- Ambient temperature: -20-55 °C
 - Relative humidity: 20%-95% (no condensation)
 - Atmospheric pressure: 700hPa-1060hPa

3 User Interface

- 3.1 System boot-up**
- Boot-up from complete shut-down in less than 25 sec
 - Shut-down in less than 15 sec
 - Restore from standby mode: about 3 sec
- 3.2 Comments**
- Supports text input and arrow
 - Support freehand marking on touch screen
 - Covers various applications
 - User customizable
- 3.3 Body mark**
- 144 bodymarks for versatile application
- 3.4 Numbers of exam mode presets: 29 system exam modes (unlimited number for user-defined ones)**
- 3.5 Screen information***

- Common info:
 - Mindray logo
 - Hospital name,
 - Acoustic power
 - Mechanical index
 - Tissue thermal index
 - ID, Last name, First Name, Middle initial, Gender, Age
 - Probe model
 - Operator
 - Focus position
 - Imaging parameters

*Not all items are listed in this part, detail info please refer to user manual

4 Imaging Parameters

4.1 Overview

- Echo-enriched Beamforming
- Up to 55296 channels
- Up to 8-beamforming

4.2 B-mode

- Display formats
- iClear™
- iBeam™
- iTouch™
- FCI
- Echo Boost™
- Image quality
- B steer: available on linear transducers
- ExFOV: available on convex, linear, and volume transducers
- Depth
- Max. frame rate
- Acoustic output power
- TGC
- Dynamic range
- Gain
- Focus number
- Focus position
- FOV
- Line density

- Persistence
- Horizontal Scale
- L/R flip and U/D flip
- Rotation
- TSI
- Gray Map
- Tint map

4.3 THI and PSH™

- Available on all types of transducer
- Patent PSH™ technology, obtains purer harmonic, better contrast resolution, higher SNR, exceptional high frequency harmonic
- iClear™ available
- Image quality

4.4 M-mode

- Display formats
- Acoustic output power
- Dynamic range
- Gain
- M sweep speeds
- M soften
- Tint maps
- Gray Map
- Edge enhance

4.5 Color Doppler Imaging

- Steer
- Image quality
- Max. frame rate
- Acoustic output power
- Gain
- ROI size/position
- Scale
- Baseline
- Wall filter
- PRF
- Packet size
- Flow state
- Smooth
- B/C align
- Priority
- Color map
- Invert
- Persistence

- Line density
 - Smart track
- 4.6 Power Doppler Imaging**
- Support directional power doppler
 - Image quality
 - Acoustic output power
 - Dynamic range
 - Gain
 - ROI size/position
 - Scale
 - Wall filter
 - PRF
 - Packet size
 - Flow state
 - Smooth
 - B/C align
 - Priority
 - Power map
 - Directional power map
 - Persistence
 - Line density

4.7 PW/CW-Mode (CW mode is an option)

- Display formats:
V2:3,V3:2,V3:1,FULL(V: vertical)
- Image quality
- Sample volume size
- SV position
- PW Scale
- CW Scale
- Baseline
- PW Steer
- Volume
- PW PRF
- CW PRF
- Gain
- Dynamic range
- Sweep speed
- Wall filter
- Invert
- Auto invert
- Angle correction
- Quick angle
- Gray map
- Tint map

- Time/frequency resolution
- Auto calc
- Auto calc cycle
- Trace area
- Trace sensitivity
- Trace Smooth
- Duplex/Triplex

4.8 LVO(option)

- Only available on P4-2s
- Dedicated left ventricle contrast imaging tool

4.9 iTouch™

- Auto image optimization
- B-mode: gain, TGC
- Color: gain
- Power: gain
- PW: baseline, scale, PRF, WF
- For L12-4s under vascular, EM Vas and carotid exam mode: iTouch also optimizes Color ROI and PW sampling line

4.10 Smart Track

- Continuously track the flow and detect the best color box position and angle in real time scanning
- Only for L12-4s under vascular, EM Vas and carotid exam mode

4.11 Zoom

- Zoom: Pan zoom (read zoom)
- iZoom: Expand the image to full screen

4.12 Quick Save

- Create a new exam mode by quickly saving current image parameter settings

4.13 iNeedle (option)

- Needle visualization enhancement
- Available on all linear transducers
- Needle steer: -50, -40, -30, -20, 20, 30, 40, 50 degrees (also supports slight adjustment in 2° increments)

5 Cine Review and Post Processing

5.1 Cine review

- Available in all modes
- Frame by frame manual cineloop review or auto playback with variable speed
- Maximum cine memory up to 32346 frames or 427s (M/PW)
- Retrospective and prospective storage are available and length is pre-settable (Prospective: Max. time 480s; Retrospective: Max. time: 120s)

5.2 Post processing

- B-mode:
 - Dyn Ra.
 - Gray Map
 - Tint Map
 - iClear
 - L/R Flip
 - U/D Flip
 - Rotation
 - H Scale
 - Echo Boost
- M-mode:
 - Speed
 - Dyn Ra.
 - Gray Map
 - Tint Map
 - Edge Enhance
- Color:
 - Baseline
 - Smooth
 - Color Map
 - Priority
 - Invert
- Power:
 - Smooth
 - Dynamic range
 - Color Map
 - Priority
 - Invert
- PW:
 - Baseline
 - Volume
 - Angle

- Speed
- Dyn Ra.
- Gray Map
- Tint Map
- Invert
- WF
- Quick Angle
- Auto Calculate
- T/F Res
- Auto Calc Cycle
- Auto Calc Parameter
- Trace Sensitivity
- Trace Smooth
- Trace Area
- CW
 - Baseline
 - Audio
 - Angle
 - Quick Angle
 - Speed
 - Dyn Ra.
 - Gray Map
 - Tint
 - Tint Map
 - Invert
 - WF
 - T/F Res
 - Angle
 - Quick Angle

6 Measurement/Analysis and Report*

6.1 Basic measurements

- Depth
- Distance
- Angle
- Area: Ellipse, Trace
- Volume :3-Distance
- Length Ratio
- Area Ratio
- Time
- Slope
- Heart Rate

- Velocity
 - D Trace
 - PS/ED
 - Automatic Doppler Spectrum Analysis
 - Heart cycle pre-settable (1, 2, 3, 4, 5)
 - Automatic real-time and retrospective tracing
 - User configurable display of items
 - Support PI, RI, TAMAX, TAMEAN, Volume Flow calculations
 - Appropriate factory setting according to applications
- 6.2 Clinical option measurement package
- Abdominal
 - Liver
 - Common Hepatic Duct
 - Iliac Diameter
 - Gall Bladder: Length, Height, Wall Thickness
 - Common Bile Duct
 - Prox Abdominal Aorta Diameter
 - Mid Suprarenal Abdominal Aorta Diameter
 - Mid Infrarenal Abdominal Aorta Diameter
 - Distal Abdominal Aorta Diameter
 - Mid Suprarenal ABD Aorta
 - Mid Infrarenal ABD Aorta
 - Abdominal Aorta Bifurcate Diameter
 - Inferior Vena Cava
 - Distal Abdominal Aorta
 - Spleen: Length, Height, Width
 - Left/Right Kidney: Length, Width, Height, Volume, Cortical Thickness
 - Bladder: Length, Width, Height, Volume, micturition volume
 - Ureter Diameter
 - Pleural: Length, Width, Height, Volume
 - Left/Right UQ: Length, Width, Height, Volume
 - Left/Right Iliac Aneurysm: Length, Width, Height
 - Pelvis: Length, Width, Height, Volume
 - Pericardial Sac: Length, Width, Height, Volume
 - Prox Aorta Aneurysm: Length, Width, Height
 - Mid suprarenal Aorta Aneurysm: Length, Width, Height
 - Mid Infrarenal Aorta Aneurysm: Length, Width, Height
 - Distal Aorta Aneurysm: Length, Width, Height
 - Aorta Bif Aneurysm: Length, Width, Height
 - Gynecology
 - Cervix: Length, Width, Width
 - Uterus: Length, Width, Height, Volume, Uterus Sum, Endometrium Thickness
 - UT-L/CX-L
 - Ovary: Length, Width, Height, Volume
 - Follicle: Length, Width, Height, Average Diameter, Volume
 - Obstetrics
 - Early OB: CRL, BPD, FL, HUM, HC, AC, Amniotic Fluid
 - 2nd- 3rd Trimester: BPD, HC, OFD, FL, AC, AF, HUM, FTA, THD, Cervix L(Trace available), Ut A, Ovarian A, Ovarian V, FHR
 - Gestational Age
 - Estimated Fetal Weight
 - Fetal Biophysical Profile
 - Gestational Age and Fetal Weight OB tables
 - Weight Percentile
 - Cardiology
 - LV Function (2D and M): Teichholz
 - Mitral Valve: MV VTI, MR VTI, MV E/A
 - Aorta Valve: AV VTI, AR DecT

- PAEDP
- RVSP
- CO(LVOT)
- Left Atrium Diam/Aorta Diam
- Urology
 - Prostate: Length, Width, Height, Volume
 - PPSA, PSAD
 - Scrotal Wall
 - Left/Right Ureter Diameter
 - Renal: Length, Width, Height, Volume
 - Bladder: Length, Width, Height, Volume, micturition volume
 - Left/Right Kidney: Length, Width, Height, Volume, Cortical Thickness
 - Left/Right Testis: Length, Width, Height, Volume
 - Left/Right Testicular: Length, Width, Height, Volume
 - Left/Right Epididymis: Length, Width, Height
 - Left/Right Epididymis: Aorta, Vein
 - Left/Right Testicular: Aorta, Vein
- Vascular
 - Carotid: CCA, ECA, ICA, Bulb, Vert A Stenosis D, Stenosis A, ICA/CCA
 - Lower Extremity Vein: C.Iliac V, Ex.Iliac V, Pop V, TP Trunk V, Peroneal V, P.Tib V, A.Tib V
 - Femoral Vein: CFV, SFV, DFV
 - TCD (Transcranial Doppler): ACA, MCA, PCA, AComA, PComA, Vertebral A, Basilar A
- Small Parts:
 - Thyroid: Length, Height, Width, Volume
 - Isthmus Height
 - Testis: Length, Height, Width, Volume
 - Left/Right Testicular: Length, Width, Height, Volume
 - Left/Right Epididymis: Length, Width, Height

- Breast Mass: Length, Height, Width, Nip. Distance, Skin Distance
- Left/Right Thyroid Cyst: Length, Width, Height, Volume
- Left/Right Epididymis: Aorta, Vein
- Left/Right Testicular: Aorta, Vein

6.3 Report

- Specific report template by application
- Editable value in report
- Images selectable
- Able to Export as PDF/RTF file

* Not all measurements are listed in this part; For more detailed information please refer to the Operators' Manual

7 Exam Storage and Management

7.1 Exam storage

- 120GB SSD. More than 74GB internal hard drive for patient data storage
- Capable to store up to approximate 354313 single frames
- Direct digital storage of single frame and cine 2D, color and Doppler.

7.2 Exam management

- iStation™ workstation dedicated for patient exam management
- Patient exam query/retrieve
- Support review of current and past exam
- New exam, Activate exam, End exam are available
- Support measurements and calculations on archived exam and images
- Export images as (BMP/JPG/TIFF/DCM/AVI format)
- Support backup/send to USB devices (hide patient information); support back up to DVD-RW media (ASUS DVD).

8 Connectivity

8.1 Ethernet Network Connection

- Cable connection
- Wireless connection: built-in wireless adaptor

8.2 DICOM 3.0

- DICOM basic (option)
 - Verify (SCU, SCP)
 - Print
 - Store
 - Storage Commitment
 - Media Exchange
- DICOM Worklist (option, HL7 supported)
- DICOM Query/Retrieve (option)
- DICOM Modality Performed Procedure Step - MPPS (option)
- DICOM OB/GYN structure report (option)
- DICOM Cardiac structure report (option)
- DICOM Vascular structure report (option)
- DICOM Breast Report (option)

8.3 iStorage(included in UltraAssist)

- Direct network storage tool between ultrasound system and personal computer

8.4 MedSight

- DICOM Basic is mandatory
- Needs to be installed on mobile terminal
- Support IOS 5.0 or above mobile terminal
- Transfer PC format images or clips from system to mobile terminal through WiFi

9 Transducers

9.1 Curved array

- C5-2s
 - Application: Abdomen, Gynecology, Obstetrics, Urology, Emergency
 - Bandwidth:

2.1-5.1MHz (-6dB)

1.5-5.6MHz (-20dB)

- Number of Elements:128
- Biopsy Guide: NGB-015, multi angle, reusable

- C11-3s

- Application: Small Part

- Bandwidth:

4-10MHz (-6dB)

3-11.2MHz (-20dB)

- Number of Elements: 128
- Biopsy Guide: NGB-018, multi angle, reusable

- V11-3Ws

- Application: Obstetrics, Gynecology, Urology

- Bandwidth:

4-10MHz (-6dB)

3-11.2MHz (-20dB)

- Number of Elements: 160
- Biopsy Guide: NGB-004, single angle, reusable

9.2 Linear

- L12-4s

- Application: Vascular, Small part, Nerve, MSK

- Bandwidth:

4.2-11.8MHz (-6dB)

3-13MHz (-20dB)

- Number of Elements: 192
- Biopsy Guide: NGB-007, multi angle, reusable

- L14-6Ns

- Application: Vascular, Small part, Nerve, MSK

- Bandwidth:

5.1-12.5MHz (-6dB)

3.5-16MHz (-20dB)

- Number of Elements: 192
- Biopsy Guide: NGB-007, multi angle, reusable

- L14-6s

- Application: Vascular, Small part, Nerve, MSK

- Bandwidth:
 - 5.1-12.5MHz (-6dB)
 - 3.5-16MHz (-20dB)
- Number of Elements: 128
- Biopsy Guide: NGB-016, multi angle, reusable
- L7-3s
 - Application: Vascular, Small part, Nerve, MSK
 - Bandwidth:
 - 3.4-6.9MHz (-6dB)
 - 2.7-8.3MHz (-20dB)
 - Number of Elements: 128
 - Biopsy Guide: NGB-007, multi angle, reusable
- 7LT4s
 - Application: Intraoperative, MSK, Vascular
 - Bandwidth:
 - 5-10MHz (-6dB)
 - 3.5-13.5MHz (-20dB)
 - Number of Elements: 128
 - Biopsy Guide: NGB-010, multi angle, reusable

9.3 Phased array

- P4-2s
 - Application: Cardiology, TCI, Abdomen, LVO
 - Bandwidth:
 - 1.7-4.1MHz (-6dB)
 - 1.3-4.7MHz (-20dB)
 - Number of Elements: 64
 - Biopsy Guide: NGB-011, multi angle, reusable
- P7-3Ts
 - Application: Cardiology
 - Bandwidth:
 - 3.1-7.2MHz (-6dB)
 - 1.9-8.2MHz (-20dB)
 - Number of Elements: 64
 - Biopsy Guide: not available

10 Peripheral Devices and Accessories (Option)

- 10.1 Black/white digital video printer
 - SONY UP-D897, MITSUBISHI P95DW-N
- 10.2 Color digital printer
 - SONY UP-D25MD
- 10.3 Graph/text printer
 - HP Deskjet 1050 J410 series, HP Officejet 7000 wide format, HP Officejet Pro 8100
- 10.4 Wireless printer
 - HP Officejet Pro 8100
- 10.5 Footswitch
 - USB port: 971-SWNOM (2-pedal)
 - USB port: 971-SWNOM (3-pedal)
 - Wireless: WFREC-1 (Receiver) + WFSW-2/WFSW-3 (Footswitch)
- 10.6 Barcode reader
 - Laser barcode scanner
 - Model: SYMBOL LS2208, DS6067

11 System Inputs and Outputs

- HDMI: 1 Port
- ECG connector: reserved
- USB: 4 USB 3.0 ports
- Ethernet: 1 port

12 Safety and Conformance

- 12.1 Quality standards
 - ISO 9001
 - ISO 13485
- 12.2 Design standards
 - EN 60601-1 and IEC 60601-1
 - EN 60601-1-2 and IEC 60601-1-2
 - EN 60601-1-6 and IEC 60601-1-6
 - EN 60601-2-37 and IEC60601-2-37
 - EN 62304 and IEC 62304
 - EN 62366 and IEC 62366
 - EN ISO 17664 and ISO 17664
- 12.3 CE declaration

TE7 system is fully in conformance with the Council Directive 93/42/EEC Concerning Medical Devices. The number adjacent to the CE marking (0123) is the code of the EU-notified body that certified meeting the requirements of Annex II excluding (4).

of the Directive.
NOTICE:
Not all features or specifications described in this document may be available in all probes and/or modes.
Mindray reserves the right to make

changes in specifications and features shown herein, or discontinue the product at any time without notice or obligation. Contact Mindray Representative for the most current information